Blefarix Toalhetes

LIMPEZA DE PÁLPEBRAS E PESTANAS

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Blefarix Wipes

HYGIENE FOR EYELIDS AND EYELASHES

The skin around the eyelids is particularly sensitive and deserves special care. In certain circumstances, excess flakiness can appear in this area, and sebum (seborrhoea) can form, making the skin look uncared for and unhygienic. Blefarix wipes are specially designed to assist optimum cleansing of eyelids. Applied locally, excess flakiness and seborrhoea are removed, intensively yet safely.

Blefarix wipes are suitable for babies, children and adults.

Directions for Use

Take a wipe out of its sachet and clean eyelids, keeping eyes closed. We recommend using one wipe for each eyelid. Massage gently to remove any particles that have become stuck; you do not need to rub hard. If necessary, the edge of the eyelid can be cleansed with one end of the wipe, taking care not to touch the eyeball.

Use whenever eyelids require cleansing.

Blefarix wipes are designed for one single use, and they are supplied sterilized, to guarantee hygienic application.

Precautions

Blefarix wipes have been formulated free from irritants; you should nonetheless avoid direct contact with the mucous membrane of the eye. In case of accidental contact, rinse thoroughly with water, taking care not to rub the area.

Presentations

Single-dose wipes impregnated with 2.5 ml of solution.

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